

MELAdes®

The new hygiene system for your practice



MELAdes® 600 Hand disinfectant



Well tolerated and effective

MELAdes 600 is a ready-for-use, alcoholic, unscented disinfectant which is rubbed into the skin and hands. Its skin-friendliness and its spectrum of activity have been designed to meet the particular requirements of medical and dental practices.

Quickly effective and practical to apply

MELAdes 800 is a material-friendly instrument disinfectant (concentrate). The extraordinary cleaning power of MELAdes 800 allows you to disinfect and clean in one working step, which in turn leads to savings in time and money.



MELAdes® 800 Instrument disinfectant



MELAdes® 700 Quick surface disinfectant



Powerful against viruses.
Gentle with surfaces and skin

For cleaning and disinfecting working areas and surfaces: The MELAdes 700 quick surface disinfectant has a pleasant odour. The formulation with the most recent active ingredients guarantees high levels of application comfort.

Efficient combination product

MELAdes 801 is a ready-for-use combination product for disinfecting and cleaning dental drills and other rotating precision instruments. It is characterised by its excellent cleaning power, its broad spectrum of activity, its effective corrosion protection and its discreet odour.



MELAdes® 801 Drill disinfectant



MELAdes® accessories All you need for a practical and economical use



Dosing pump

With the dosing pump, a comfortable extraction of the preparation is guaranteed, e.g. in order to fill the bur container. By using the reducers included in the scope of delivery, an extraction of a specific quantity of 30 ml is enabled while the dosing of the concentrates is simplified. The dosing pump can be fitted onto the 5-litre- and the 10-litre-cans.



Wall dispenser

The wall dispensers for the MELAdes 600 hand disinfectant are equipped with a long operating lever, which allows for a comfortable operation with the elbows. Both the arm lever as well as the stainless-steel pump can be sterilized in the autoclave. The wall dispenser is available for the 500 ml-dispenser bottle as well as for the 1-l-dispenser bottle.



Bur container

The bur container has a volume of 150 ml and is used for disinfecting and cleaning rotating instruments.



Instrument bath

The instrument bath has a volume of 3 litres and is used as container for the manual cleaning and disinfection of instruments.



Can tap

With the can tap, an ergonomic and comfortable extraction of larger quantities from the 5-litre- and 10-litre cans is possible. The use of the can taps is particularly recommended when preparing a new solution of the MELAdes 700 quick surface disinfectant with fleece cloths.

Overview over the areas of application and spectra of effectiveness

Product	Cleaning	Disinfection	Bactericidal	Tub effective	Yeasitcidal	MRSA	active against all enveloped viruses (Vaccinia, BVDV, HBV/HCV/HIV)	Adeno viruses	Papova/Polyoma viruses	Noro viruses	Rota viruses	Concentrate	Ready for use	Alcohol	Formaldehyde-free	Aldehyde-free	Phenol-free	Suitable for ultrasonic bath	Listed by the DGHM (German association for hygiene and microbiology)	Concentration	Soaking time according to the DGHM stipulation	Remarks
MELAdes® 600 Hand disinfectant:	x	x	x	x	x	x	x			x	x		x	x	x	x	x	x	x	conc.	30 sec.	Hand disinfection: conc. 30 sec. Hand and forearm disinfection: 1.5 min. active against all enveloped viruses (Vaccinia, BVDV, HBV/HCV/HIV): 30 sec. Rota viruses: conc. 15 sec. (for the hygienic disinfection of hands, a soaking time of at least 30 seconds is to be adhered to) Noro viruses: conc. 1 min.
MELAdes® 700 Quick surface disinfectant	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	x	conc.	* 1 min.	* listed in the column 5 minutes bactericidal incl. MRSA and TB, yeasitcidal active against all enveloped viruses (Vaccinia, BVDV, HBV/HCV/HIV): 30 sec. Noro viruses: 5 min.
MELAdes® 800 Instrument disinfectant	x	x	x	x	x	x	x	x	x			x		x	x	x	x	x	x	* 1% 2% 3%	* 60 min. 30 min. 15 min.	Bactericidal incl. MRSA and TB, yeasitcidal 1% 30 min. 3% 15 min. Enveloped viruses incl. HBV, HIV, HCV, Vaccinia, BVDV Adeno viruses 2% 60 min. 4% 30 min.
MELAdes® 801 Drill disinfectant	x	x	x	x	x	x	x	x	x				x	x	x	x	x	x	x	conc.	* 5 min	Bactericidal incl. MRSA and TB, yeasitcidal

*High organic load
Use disinfectants safely. Always read the label and product information before use.

We have been a partner for practices and clinics for more than 60 years

Autoclaves, washer/disinfectors, sealing devices, water treatment systems, documentation software and now also disinfectants.

MELAG –
everything for instrument treatment and practice hygiene.



MELAdes is manufactured in accordance with the following regulations and standards: 93/42/EEC Class IIb Medical Devices (Medical Devices Directive)*, 98/8/EC (Biocide Directive), ISO 9001 and ISO 13485 (Quality Management/Certification).

* only applicable to MELAdes@700, 800 and 801

Further information on this subject, our products as well as current questions can be obtained from: www.melag.com



MELAG Medizintechnik oHG
Geneststraße 6-10
10829 Berlin



CE 0197

MELAG
competence in hygiene

MELAdes® Systematic hygiene



www.melag.com
Quality – made in Germany

MELAG
competence in hygiene